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The SONIMAGE HS1 Diagnostic Ultrasound System: Triad Tissue Harmonic Imaging

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Abstract

Ultrasound is a real-time, less-invasive imaging modality that can be used in a wide range of clinical situations. In July, 2014, Konica Minolta launched the SONIMAGE HS1, the first ultrasound system ever developed by the company. This system provides resolution capable of detecting structures as small as just several hundred microns, making it ideal for use in musculoskeletal and superficial applications.

The SONIMAGE HS1 was developed to realize high sensitivity and ultra-wideband linear probe features. Advancements in the SONIMAGE HS1 include such transducer design improvements as multi-layered acoustic matching layers and a low attenuation acoustic lens, as well as optimized material characteristics of these components, and fine processing technology. The L18-4, the system's high-frequency linear probe, has overcome many common trade-offs between sensitivity and frequency bandwidth.

In addition, we developed a unique signal transmitting/receiving technology, Triad-THI (triad tissue harmonic imaging), to generate three separate harmonic components that cover the entire receiving spectrum of the ultra-wideband probe.

As a result of these developments, the SONIMAGE HS1 system is able to balance penetration and resolution for an optimum image.

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Fig. 1 The SONIMAGE HS1 ultrasound imaging system and B-mode image by linear probe.

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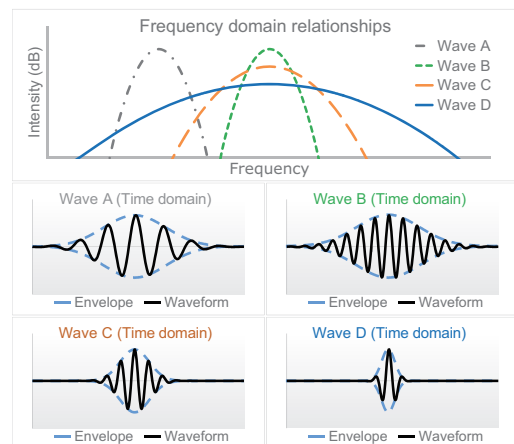
Fig. 2 Relationships between spectrums in the frequency domain (top) and the waveforms in the time domain (bottom). Axial resolution improvement needs wide frequency bandwidth.

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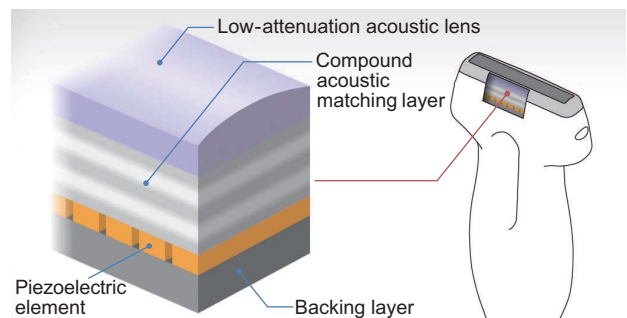
Fig. 3 New high frequency linear probe (top) and structure of the ultrasound transducer (bottom). The compound acoustic matching layer and acoustic lens were developed using Konica Minolta's material technology.

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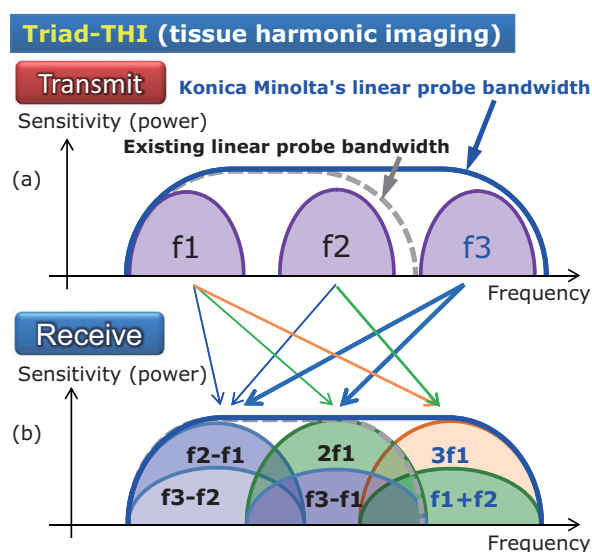
Fig. 6 Components of transmission and receiving frequencies. (a) Transmitted fundamental components and (b) received harmonic components. Conventional frequency band signal is increased by f_3 's differential harmonics. Receiving bandwidth is expanded by higher frequency harmonics.

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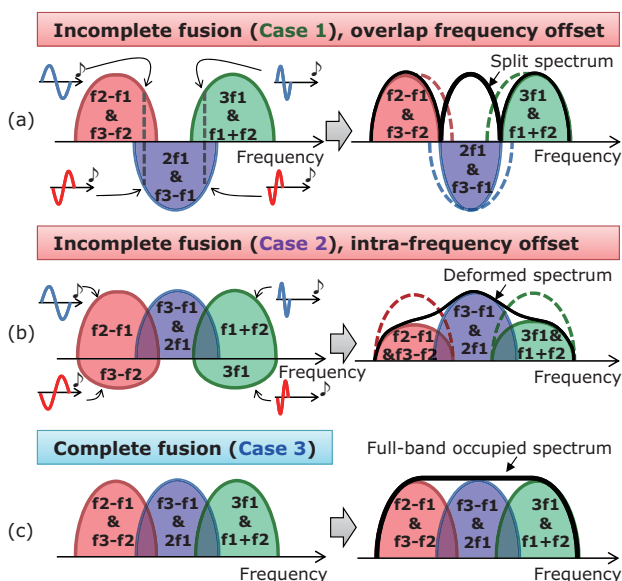
Fig. 7 Harmonic spectrum fusion resultant from phase difference. Complete phase fusion is important for high resolution. (a) Incomplete fusion (Case 1), overlap-frequency offset. (b) Incomplete fusion (Case 2), intra-frequency offset. (c) Complete fusion (Case 3).

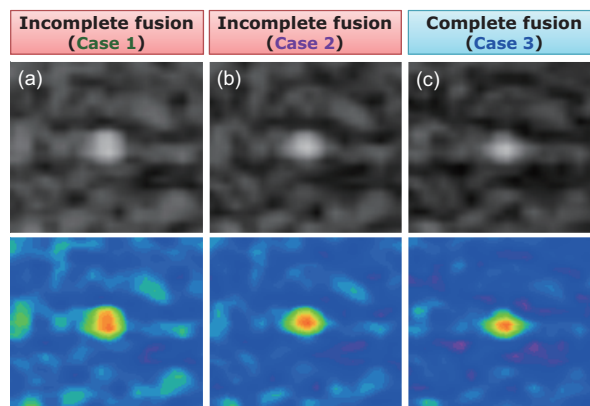
Fig. 8 Phantom images corresponding to the cases of fusion in Fig. 7.

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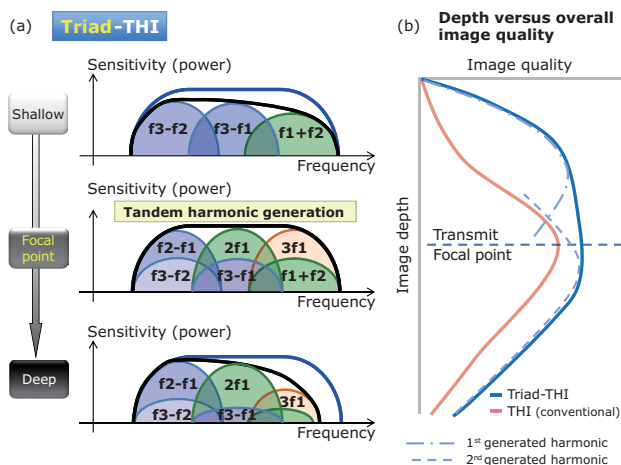
Fig. 9 Imaging depth versus (a) distribution of generated harmonic components, and (b) overall image quality with the Triad-THI. The high-quality image area is expanded through two-step harmonics generation (tandem harmonic generation).

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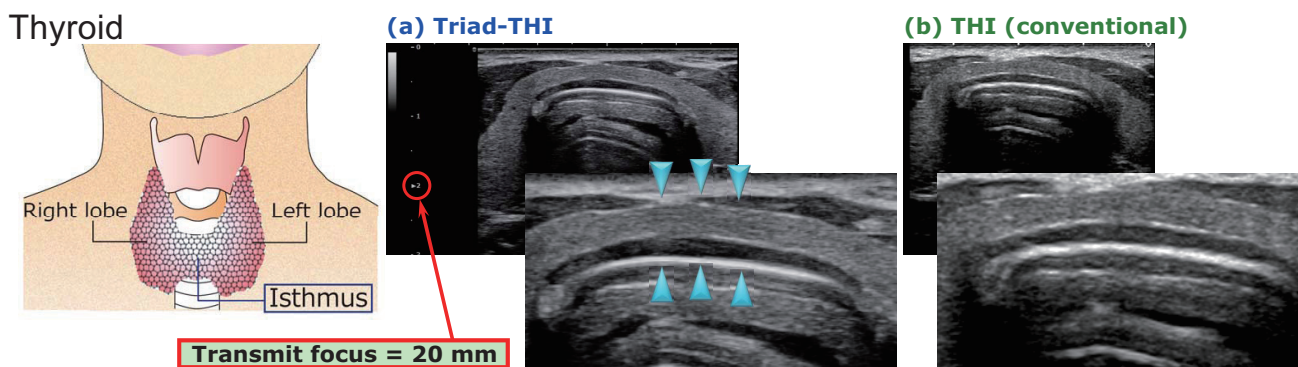
Fig. 10 Schema and clinical images of the thyroid.

- (a) Triad-THI images — Excellent tissue separation (sharp and with high contrast) at shallow isthmus (arrows) (focus = 20mm).
 (b) Conventional THI image (Konica Minolta system) — Shallow area is unclear (focus = 20mm).

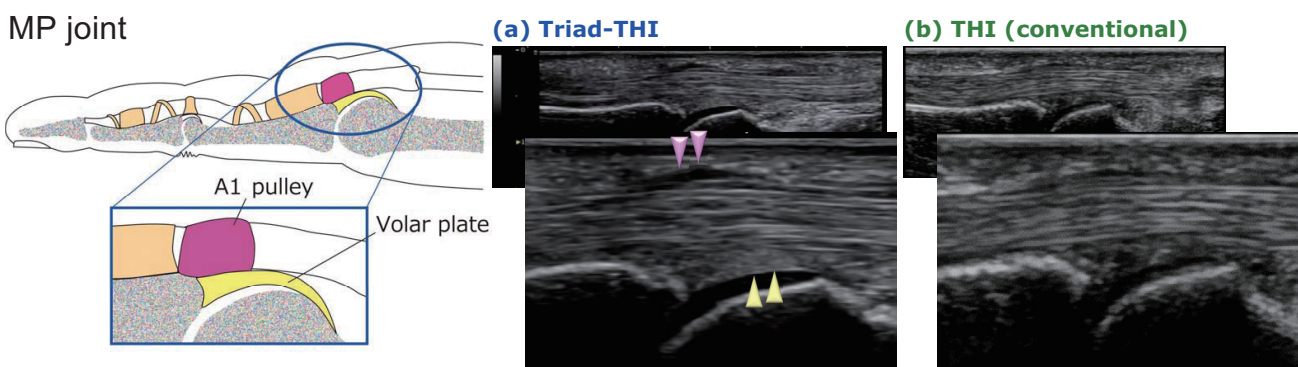
Fig. 11 Schema and clinical images of MP (metacarpophalangeal) joint

- (a) Triad-THI image — A1 pulley (pink upper arrows) is described clearly, the sharp boundary between the bursas and tissue is excellent (yellow lower arrows), and the bone surface is smooth.
 (b) Conventional THI image (Konica Minolta system): surpassed in quality by Triad-THI image.

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